



NEODRYS®

Stop wasting money on pads and rolls that either fall apart or get saturated quickly.
When you add up the reduction in botched procedures and wasted time - **NEODRYS cost less!**



Absorb

- Super absorbent core—out performs messy cotton rolls and other absorbent pads.
- No messy leaks, even with the core fully saturated.


Protect

- Protects procedures from moisture contamination from the parotid gland for up to 15 minutes.
- Stays in place yet removes easily with water spray.
- Protective backing plus poly stiffener provide cheek retraction and protection.

Reflect

- Choice of Original white or Reflective backing brightens the oral cavity, while retracting the cheek for an improved view.

NeoDrys Features

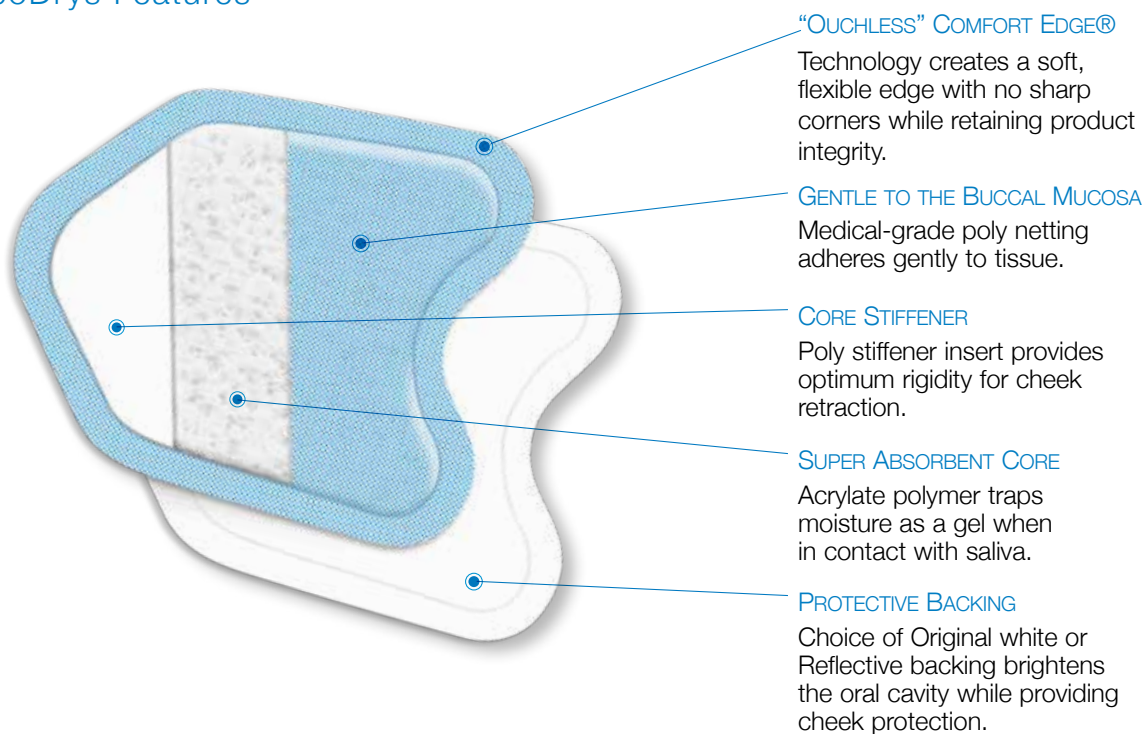


Patents:
US7425664
EP1676552

Method of Application


Select NEODRYS size which adequately covers the buccal mucosa. Insert as shown with color side against cheek and point to back of mouth. In a few seconds, NEODRYS will begin to adhere to the tissue and stay in place.

Important: To remove without irritating tissue, release adhesion with ample water spray to the buccal side of the NEODRYS.


Color to Cheek

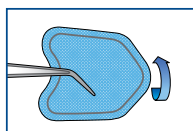

Point to Back

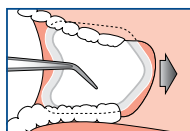

Spray to Release

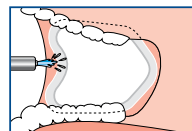

ABSORB, PROTECT & REFLECT

Easier & More Effective Treatments.

- Sealants
- Composites
- Cementation
- Impressions
- Occlusion Marking
- Orthodontics
- CAD/CAM Prep
- Amalgam Preps
- Crown & Bridge
- Laser Dentistry
- Fluoride Applications
- Core Buildups

Color-Coded Sizes

NEODRYS are color coded for quick identification. Available in two sizes: Large (blue) and Small (yellow). Packaged in convenient, easy-to-handle packs of 50.



Choice of Backing

Available in Reflective light-enhancing, mirror-like finish or original white backing.

White Backing



50 per pack

Order No.

545655 - Small (Yellow)
545656 - Large (Blue)

Reflective Backing



50 per pack

Order No.

545658 - Small (Yellow)
545659 - Large (Blue)